


Mounting to a Permobil® Chair

Required Items:

- POSAlinc® 360 or POSAlinc® Original Track-Mount Knee Adductor
- Permobil® Track Insert Kit (Part No. 11143) – *sold separately*
Order 1 Permobil Track Insert Kit per each Knee Adductor

1. Unscrew the standard Track-Mount Assembly from the Knee Adductor.
2. Remove the standard ¼-20 SHCS screws, track nuts, and washers from the Track-Mount Assembly.
3. Use the Permobil Track Insert Kit, M6 x 14mm SHCS (E) screws and M6 Hex Nuts (H), in place of the standard hardware.
4. Slide the Hex Nuts into the Permobil UniTrack (G) and position.
5. Re-assemble the track system mounting post (F) using the M6 screws. Another alternative to what is illustrated below: the track system mounting post may also be mounted utilizing both UniTrack “T-Slots”.
6. Tighten all fasteners.

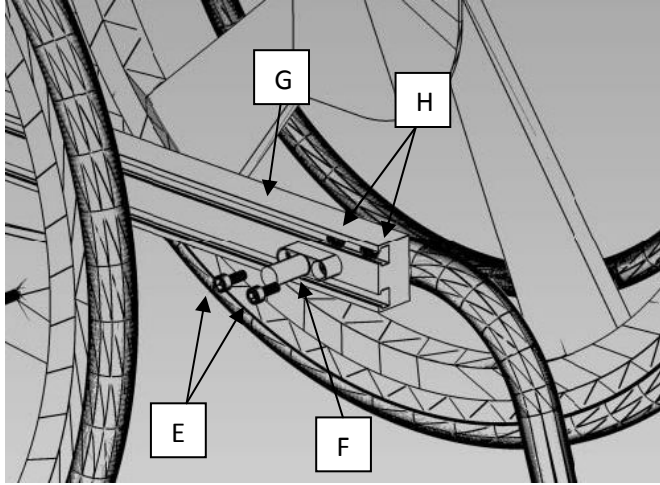

Next: Follow Instructions to *Mount the Remaining Device Components* in the Knee Adductor Instruction Booklet

Permobil® is a registered trademark of Permobil

Form AR-077