



Rear-Pull, Front-Pull, & Dual-Pull on One Support

Convert your *front-pull* AirLogic Anterior Trunk Support to *rear-pull* or *dual-pull* with the AirLogic Dual-Pull Strap Kit

- Adds rear-pull straps
- Provides the option to move the top buckles up to 6" away from the pad – so that they sit behind the back
- Adds an additional 6" in length to the top straps

Versatility to Accommodate More Positioning Needs

Option 1: Dual-Pull

To offer the option to adjust with both Rear-Pull and Front-Pull top straps, simply replace the standard top straps with the Dual-Pull Strap Kit.



Option 2: Rear-Pull

To add Rear-Pull straps, while eliminating the Front-Pull option:

Step 1 Replace the standard top straps with the AirLogic Dual-Pull Strap Kit

Step 2 Pull the Front-Pull straps up so that they sit next to the webbing loop

Option 3: Position Buckles Behind the Back

To move the top buckles away from the pad – so that they sit behind the back:

Step 1 Replace the standard top straps with the AirLogic Dual-Pull Strap Kit

Step 2 Pull the Front-Pull D-Rings through the webbing loop.

Step 3 Move the D-Rings up so that they sit next to the Side Release Buckles.



Use the Rear-Pull straps for adjustment

No specific maintenance is required. However, all fasteners should be routinely checked for specified tension and basic cleaning will maintain good hygiene.

We reserve the right to continue to provide state-of-the-art products. Therefore, specifications may be changed without prior notice. Adaptive Engineering Lab, Inc shall not be responsible or liable for misapplication or misuse of our products. We recommend experienced professionals be consulted in selecting and applying our products.

© 2014 by Adaptive Engineering Lab, Inc. All rights reserved.

Registered Trademarks of Adaptive Engineering Lab, Inc: ARCUfit, POSAfit, POSAlinc, QuickPlus VERSALock, Dynaform Form AR-078